


MORE THAN A GLOW. A SKIN TRANSFORMATION.

DiamondGlow™ A next-level skin resurfacing technology that does more than just skim the surface.


This powerful treatment simultaneously exfoliates, extracts, and treats the skin for results that last. It infuses the skin with serums at the moment of exfoliation when pores are most receptive, maximizing every second of the treatment, for proven results.

3X1™ TECHNOLOGY IN ONE, SIMPLE STEP

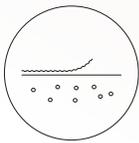


EXFOLIATION

Precise diamond tip helps remove dry, damaged skin

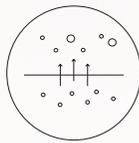


EXTRACTION

High-power suction deep cleanses the skin's surface and pores

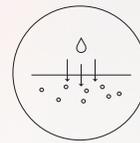

INFUSION

High level of skin receptivity allows SkinMedica® Pro-Infusion Serums to deliver professional-grade results

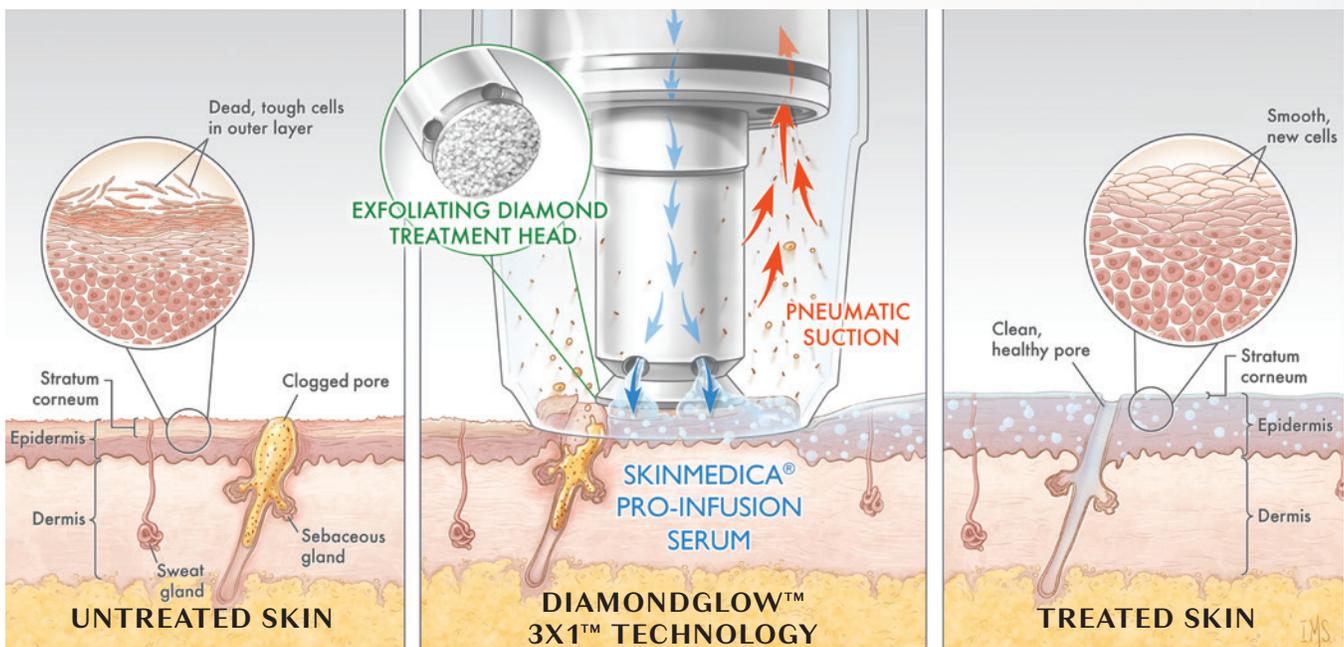

GLOWING CLINICAL RESULTS

DIAMOND
GLOW™

VISIBLE RESULTS IMMEDIATELY POST-TREATMENT¹



89% of patients experienced improvements in **overall dryness**



90% of patients experienced improvements in **radiance**

^(P) Note: These are patient assessments.


PATIENT SATISFACTION IS HIGH¹

IMMEDIATELY FOLLOWING THE
DIAMONDGLOW™ TREATMENT:

100%

of patients noted:

- My skin feels firmer
- I will be going back for more treatments
- My skin is more radiant
- It was a deep clean

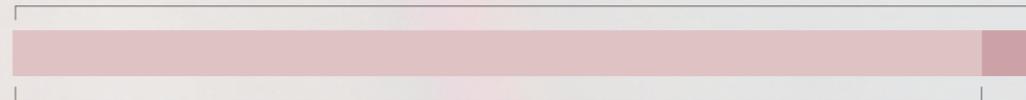

95%


of patients noted*:

- It was an enjoyable treatment
- My skin feels more plump

^(P) Note: These are patient assessments.

CONTINUED IMPROVEMENT AT 72 HOURS

A continued decrease in the appearance of dryness, lines, and roughness

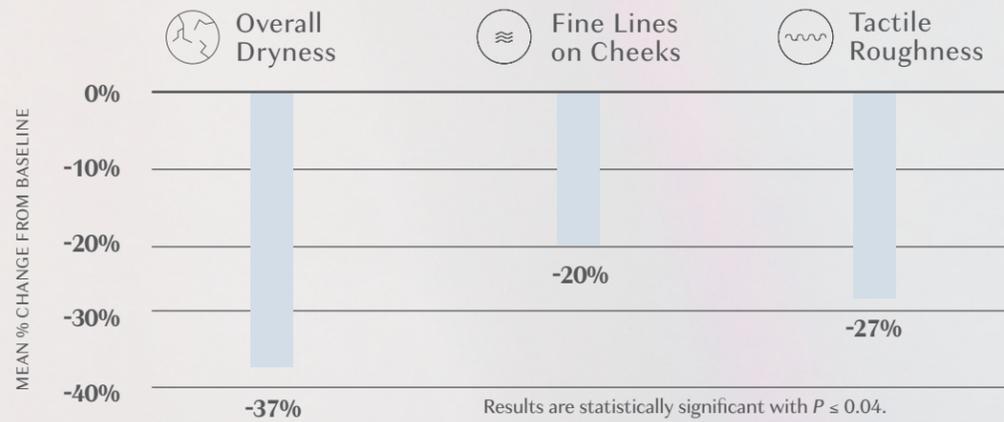

Results are statistically significant with $P \leq 0.04$.

⁽⁺⁾ Note: These are healthcare provider assessments.

Indication and Use

The DiamondGlow™ device is indicated for general dermabrasion of the skin and also delivers topical cosmetic serums onto the skin.

Please see the back page for Important Safety Information and the SkinMedica® Pro-Infusion Serums disclaimer.

*n = 20.


LONG-TERM IMPROVEMENT¹

DIAMONDGLOW™ + SKINMEDICA® TAKE-HOME REGIMEN

DIAMOND
GLOW™

Keep the glow going with targeted take-home regimens your patients can easily fit into their daily routine between DiamondGlow™ treatments to optimize and enhance their in-office results.

MEAN % CHANGE FROM BASELINE SHOWS
SIGNIFICANT IMPROVEMENTS:

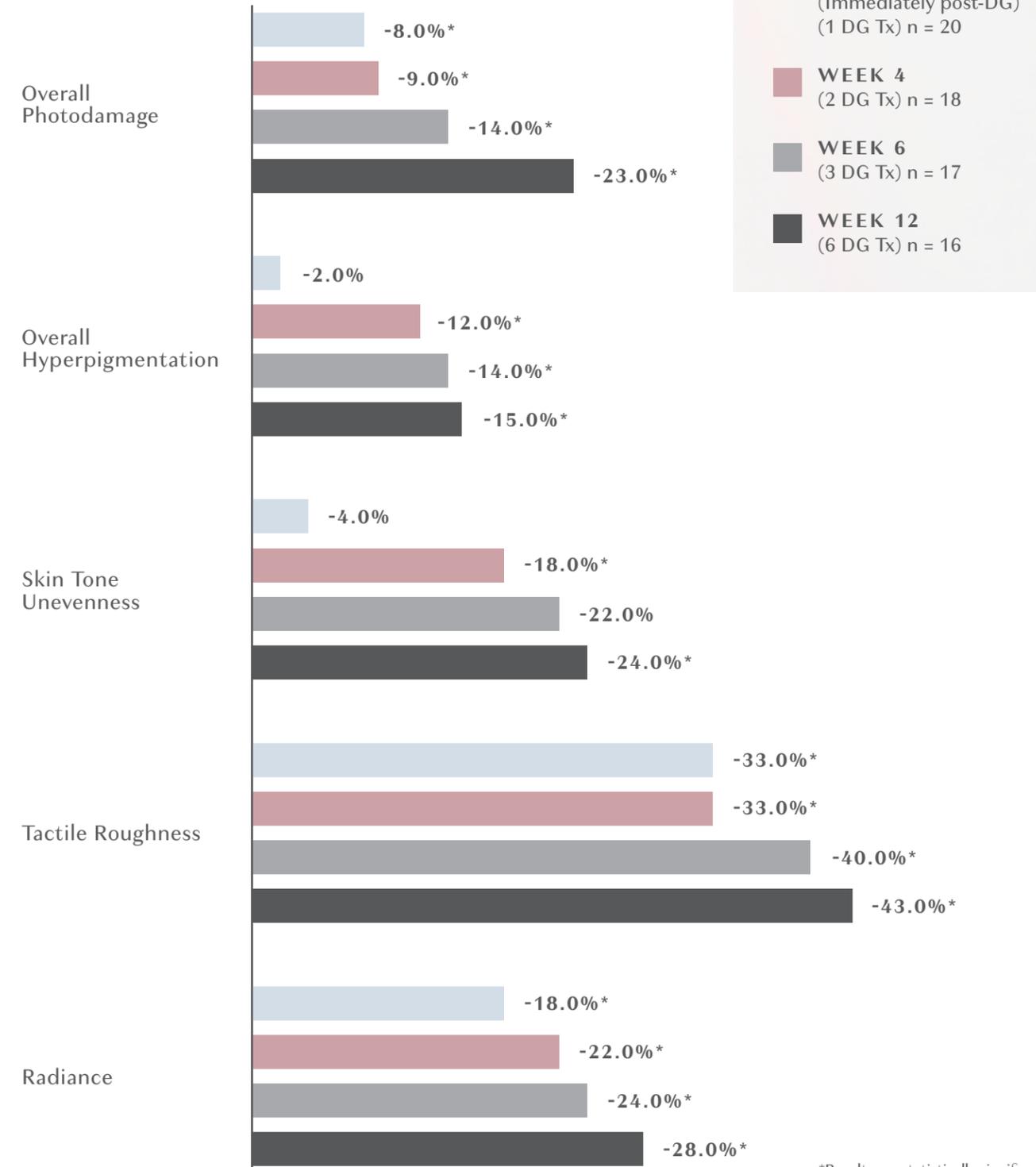

*Results are statistically significant with $P < 0.003$.

⁽⁺⁾ Note: These are healthcare provider assessments.

RESULTS ARE INSTANT AND LONG LASTING

DIAMOND GLOW™

INSTANT RESULTS



Unretouched photos. One treatment with SkinMedica® Pore Clarifying Pro-Infusion Serum (on face and neck) and SkinMedica® Skin Brightening Pro-Infusion Serum (around the eyes and lips). **Individual results may vary.**

CONTINUED IMPROVEMENT



Unretouched photos. Results after 4 weeks following 2 DiamondGlow™ treatments with SkinMedica® Pore Clarifying Pro-Infusion Serum administered 2 weeks apart and a curated SkinMedica® skincare regimen. **Individual results may vary.**



Unretouched photos. Results after 6 weeks following 3 DiamondGlow™ treatments with SkinMedica® Skin Brightening Pro-Infusion Serum administered 2 weeks apart and a curated SkinMedica® skincare regimen. **Individual results may vary.**

DiamondGlow™ Important Safety Information

DiamondGlow™ is contraindicated in patients who have compromised skin quality including but not limited to, sunburned, chapped, irritated or broken skin, open wounds, active, weeping acne, cold sores, or herpetic ulcers. Ask your patient if they are pregnant or lactating or if they have any medical conditions, including allergies, and usage of topical medication on the area to be treated.

Typical side effects include a scratchy, stinging sensation during the treatment and temporary tightness, redness or slight swelling after the treatment. Rare serious side effects may also occur and include severe skin irritation and allergic reactions. Cease use of the device immediately if any of these serious side effects are observed.

Reference: 1. Data on file at SkinMedica®

Patients should be advised to use a sunscreen with a sun protection factor of 30 or higher following treatment.

Consult the DiamondGlow™ User Manual for a complete list of Contraindications, Warnings, Precautions, and potential side effects.

SkinMedica® Pro-Infusion Serums Disclaimer

SkinMedica® Pro-Infusion Serums are intended to meet the FDA's definition of a cosmetic product, an article applied to the human body to cleanse, beautify, promote attractiveness, and alter appearances. These products are not intended to be drugs that diagnose, treat, cure, or prevent any disease or condition.

**Allergan
Aesthetics**
an AbbVie company

© 2020 AbbVie. All rights reserved. DiamondGlow™ and its design are trademarks of Allergan, Inc., an AbbVie company.
AllerganAesthetics.com DiamondGlow.com 138096-v2 10/20 006679